biöwalker®

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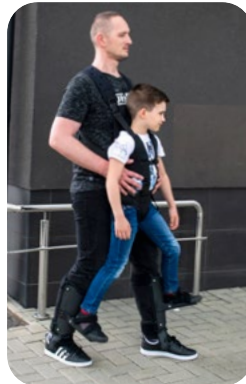
biöwalker® is an active upright device for children with mobility impairment. The upright position is achieved by Patient and Parent or Carer. The product is made up of three parts: an adult vest, a child hip harness and calf shells with sandals. Hip harness maintains the Patient in the correct position and reduces the load of the lower limbs. The Patient's feet are placed on the comfortable sandals made of ABS. These sandals are assembled to calf shells of the Carer or Parent. They provide active, anatomic gait pattern with proper coordination between upper and lower limbs. You can adjust the height of the sandals.

biöwalker® reflects the physiological Patient's gait pattern and improves cerebral neuroplasticity and helps to restore psychomotor functions. Our upright mobility device prevents against the muscle weakness and contractures, increases the bones density and improves the vascular and respiratory system. What is more, **biöwalker**® provides Patient's independence and peer socialization. The device may be used only on harden, even surface.

REH4MAT® products are innovative and carefully manufactured of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact us by e-mail:

biuro@reh4mat.com

CONTENTS:

1. Purpose of use 19
2. Product description 19-20
3. Safety rules 21-22
4. Care and Maintenance 22
5. Using 23-30

1. PURPOSE OF USE

biöwalker® is an active upright device which allows Patient to maintain standing position and walking together with Parent or Carer. The device is good for children who cannot stand and transfer by their own. **biöwalker**® is recommended for children from age 1 (physiological walking milestone). The product is made up of three parts: an adult vest, a child hip harness and calf shells with sandals.

INDICATIONS:

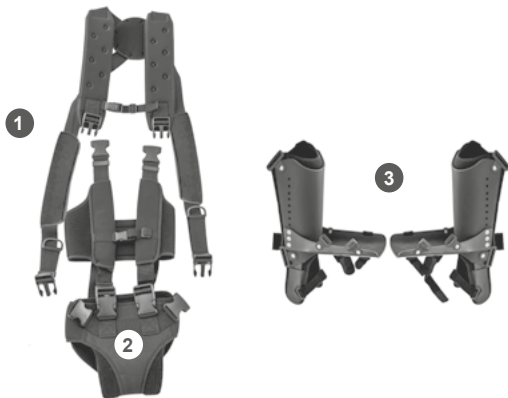
- Cerebral palsy (CP)
- quadriplegia/tetraplegia
- dyskinesia (athetosis, chorea)
- dystonia
- spinal bifidia
- paraplegia
- small muscles contractures
- multiple sclerosis (SM)
- abnormal gait pattern

CONTRINDICATIONS:


- severe muscles and tendons contractures
- sprain/strain in the lower limb area
- severe body asymmetry (high degree of scoliosis, pelvic and/or spinal deformations)
- Lower back pain of the Carer
- Pregnancy of the Carer
- Patient with unstable medical
- condition requiring fast repositioning for treatment

WARNING! If there are any doubt about Child's or Carer's ability to use the **biöwalker**®, please consult doctor or therapist.

2. PRODUCT DESCRIPTION

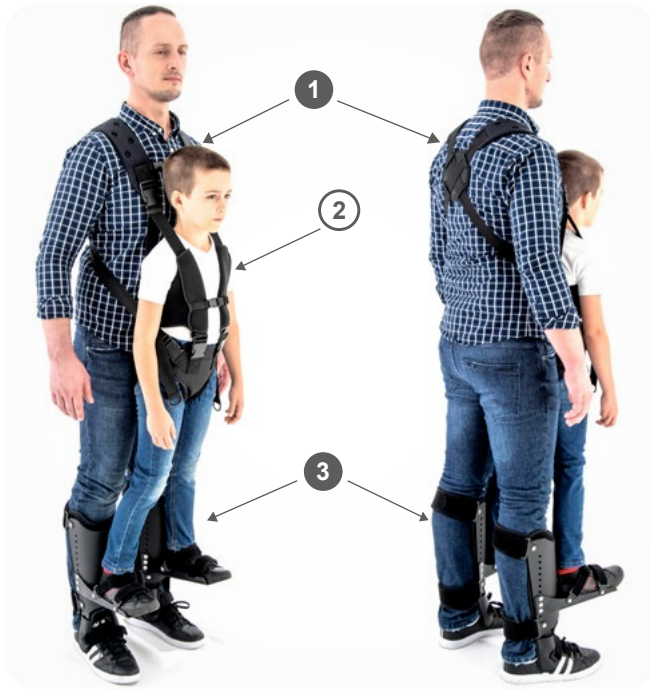1. Adjusted adult vest
2. Trunk and hip harness for Patient
3. Calf shells with sandals

CARER

Mature person who knows this user manual. It can be **PARENT OR CARER**. This person is staying physically and mentally fit. Due to **CARER'S** walking, the **PATIENT** achieves natural gait pattern. Functional design of the **biowalker** provides parent-child proximity and improves rehabilitation process.

PATIENT

Child in age 1 or more with maximum weight 40 kilos (88 lb). **PATIENT'S** feet are assembled on the sandals which moves together with **CARER**.

The **PATIENT'S** height cannot be below the clavicles of the **CARER**.

3. SAFETY RULES

WARNING! Read this instruction carefully before use and keep for future reference. In other case you may hurt yourself and the Child.

WARNING! You can use the device only on the even, hardened surface.

WARNING! Only mature person, staying physically and mentally fit, may be a Carer.

WARNING! Carer cannot be under the influence of psychoactive substances, drugs or alcohol.

WARNING! If some problem of Patient or Carer occur during using the device (fainting, fatigue), stop using the products and consult your doctor immediately.

WARNING! Using the **biöwalker** on stairs, may result in injury and is not recommended.

WARNING! Always check the product before using. If any signs of damage are observed, do not use it! Do not use the **biöwalker** if any part is broken or missing. It may hurt you and the Child.

WARNING! Always ensure all straps are securely fastened. Check if straps do not unfasten due to Patient's and Carer's weight. Improper assemble may hurt the users after slipping.

WARNING! Do not change the design of this device. Do not attach, remove, sew and cut any elements. Doing any changes in this product or using not in accordance with instructions increases the risk of hurt the users.

WARNING! Adjust length of straps and tapes only in the sitting position. Do not adjust the products while standing! It may risk in severe injury of the users.

WARNING! Please follow instruction in "USING" (point 5 in this manual). Putting and removing the **biöwalker** is possible only in the sitting position. While sitting or standing we recommend to use handrail or handle for better balance.

WARNING! Keep the device clean. Please follow the instructions in "CARE AND MAINTAINANCE" (point 4 of this manual).

WARNING! Ensure Patient's hip harness and Carer's hip girdle are on the same level. Maintaining the hip harness above the Carer's pelvis may result in severe lower back pain of the Carer.

WARNING! Recommended shoes while using the **biöwalker** are trainers (sneakers) for safety and stability reasons. Shoes with heels, large boots, sandals or slippers should not be worn.

WARNING! The Patient's body weight should be distributed on both feet. Patient cannot "hung" in the device. If the Patient's feet do not have a full contact with sandals, we do not recommend to use the **biöwalker**.

WARNING! Straps cannot be fasten too tight or too loose. They cannot be curled or twisted.

WARNING! Always ensure the device is fitted and assembled to Patient and User properly before using.

Both, Patient and Carer, should breath freely and feel comfortable in the device.

WARNING! The **biöwalker** should not have directly contact with skin. Redness on Users skin may be visible and is expected on weight bearing points. If however this redness persists, please review the set-up of the product to ensure it is not too tight or too loose. If the redness still visible, stop using **biöwalker** immediately.

WARNING! Maximum Patient weight is 50 kilos (110 lb). Never exceed the maximum load!

WARNING!

If the device is assembled properly and Patient is above the Carer's clavicles it means the Patient is too high in relation to Carer and cannot use the **biöwalker**[®] (fig.1)

WARNING!

Manufacturer takes responsibility only in case of purchasing the product in authorised distributor or medical shop.

WARNING!

Manufacturer takes responsibility only for hidden faults and technical defects of the product which occurred during a proper use of the device.

WARNING!

Use the **biöwalker** only in accordance with the intended use. In other case using the product may result in severe injury and complications

WARNING!

For single Patient use only. Using the device not according to the manual may cause serious complications.

fig.1

4. CARE AND MAINTAINCE

Aadult vest, a child hip harness and soft fabric of calf shells (lining, straps):

- wash by hands in warm water with soap in temperature of 30°C,
- Sluice carefully ,
- Do not iron and do not chlorinate,
- Do not dry clean and do not tumble dry,
- Protect from moisture,
- Protect from high temperature and dry away from sources of warmth.

Sandals:

- Do not wash and do not sluice,
- Wipe with a clean, soft cloth,
- Use a mild cleaning agent like soap.

5. USING

STEP 1 - PUTTING THE HARNESS ON PATIENT

1.1. Unfold harness on the even surface e.g. bed or couch.

1.2 Lay Patient on the unfolded harness. Remember to place pelvis, trunk and chest in the correct position. Look at picture (fig.2).

WARNING! The Upper part of the hip harness should be on the level of anterior superior iliac spine of Patient.


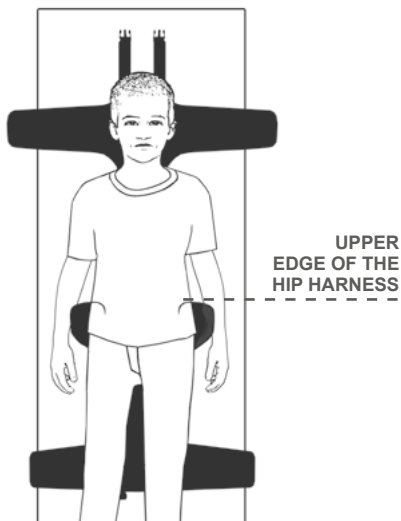
fig.2

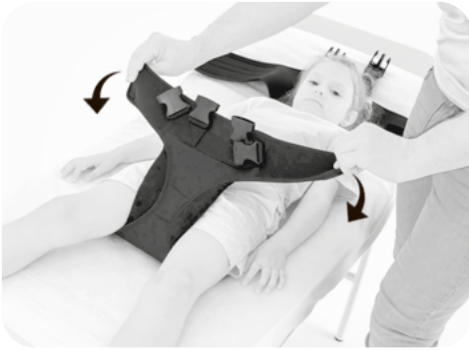Drawing how to place Patient on the unfold harness correctly.

1.3. First, fasten the hip belt.

1.4. Fasten the Velcro closures.

1.5. Next, fasten the buckles and connect the trunk belts with hip harness.

Adjust the required length of the chest's belts.

1.6. Connect two chest belts with the middle buckle. Adjust the required length of the middle strap if it is necessary.

1.7. Roll the Patient gently and closure the Velcros. Remember to support the pelvis and scapulas.

WARNING! Remember that chest belts should not go under the trunk belt.

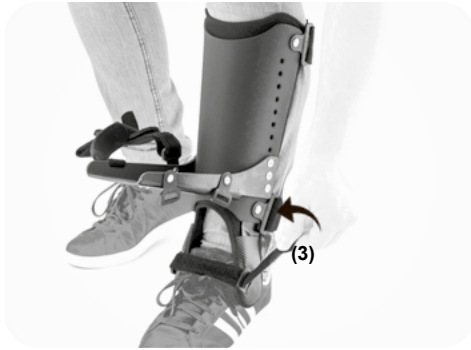
STEP 2 - PUTTING THE SANDALS

2.1. Calf shells (el.3) should be place in the shin area.

2.2. Fasten the Velcro straps on the back of the calf. First fasten the lower strap (1), then fasten the upper one (2).



2.3. Fasten the strap which goes on your shoe.

2.4. Sandals are ready to use.

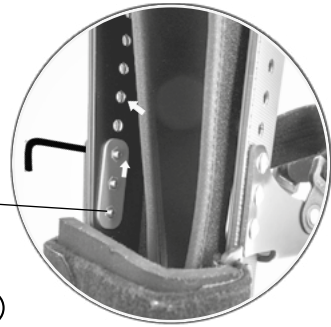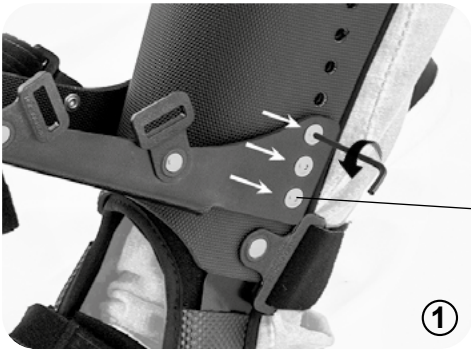SANDALS HEIGHT ADJUSTMENT:

WARNING! If you need to change the height of the sandale, unscrew 3 socket screws on the left and right side.

Change the place of the thread washer and adjust the required height.

Adjust the required position of the sandals.

Screw the socket screws firmly.

STEP 3 - CONNECTING THE CHILD TO THE CARER

3.1. Place the straps but remember to place round element (1) between your scapulas.

3.2. Fasten the front middle strap.

3.3. . Place the child on your knees.

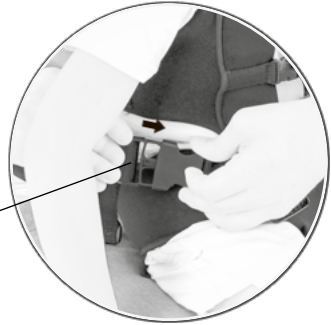
3.4. Fasten the upper buckle to the child's harness.


3.5. Fasten the lower buckle to the hip harness.

3.6. Place the child's feet on the sandals.

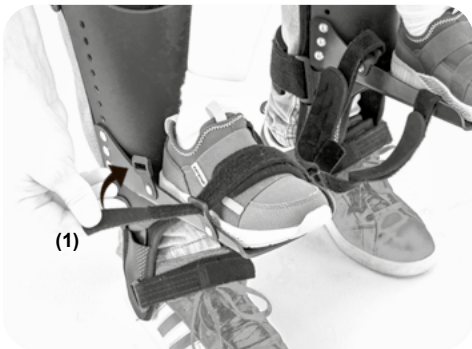WARNING! Before you fasten the straps, remember to check if the child's heels touch the calf shells.

(1)

(2)

3.7. Fasten the foot straps. First, route the metatarsal strap (1) through the buckle, Do the same activity with the upper strap (2). Do it on both sides.

WARNING! You may stand up with the child only in case of proper connection, adjusting and fastening each element of the **biöwalker**. If the straps and belts will be fastened improperly (not according to user manual), the risk of falling, hurting and injury is very high!.

3.8. Stand up gently. Do not move instant. We recommend to use handrail or handle for better balance.

1. Fit the device by pulling the harness straps.
2. If it is necessary, adjust your shoulder belts.
3. Your shoulder belts are equipped with special soft protection. Place it on your pelvis (on the iliac crest). It will reduce the pressure on your bony prominences (3).

Wait for a few minutes and check the reaction of the child.

WARNING! If you will see any worrying symptoms such as dizziness, worst contact or fainting, do not continue the upright position and stop using the device. Remove the Patient from the **biowalker** and contact your doctor or therapist. If any worrying symptoms occur, you can start therapy after few minutes.

3.9. Products is reasy to use (front and rear view).

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