walkër®

ORTEZA TYPU HKAFO
HIPS-KNEE-ANKLE-FOOT ORTHOSIS

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TABLE OF CONTENTS:

1. Purpose.....	3
2. Product description.....	3
3. Size table.....	4
4. Safety rules.....	4
5. Maintenance and hygiene.....	5
6. Setting up.....	6

1. PURPOSE

INTENDED USE:

- cerebral palsy (CP),
- quadriplegia,
- dyskinesia (athetosis, chorea),
- dystonia,
- spina bifida,
- paraplegia,
- moderate muscle contractures (especially of the lower limbs),
- multiple sclerosis,
- abnormal gait pattern.

CONTRADICTIONS:

- advanced muscle and tendon contractions,
- fracture / sprain in the lower limbs,
- strong asymmetry of the body (advanced scoliosis, deformities in the spine and pelvis),
- no medical recommendation to stand for the user.

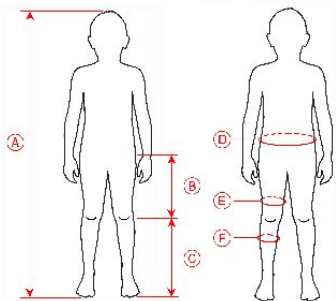
ATTENTION! If you have any doubts about the appropriateness of using the product by a child, consult your doctor or physiotherapist.

2. PRODUCT DESCRIPTION

3. SIZE TABLE

Symbol	Measurement	Size 1 [cm]	Size 2 [cm]
A	Height	122 - 140	140,5 - 158
B	Distance between knee joint interstice and groin	27 - 30	30,5 - 34
C	Distance between knee joint interstice and foot	30 - 38	38,5 - 49
D	Waist circumference	55 - 70	58 - 75
E	Thigh circumference	25 - 38	27 - 40
F	Calf circumference	21 - 27	22 - 30
	Maximum load	30 kg	40 kg

4. SAFETY RULES

ATTENTION! Before use, read the user manual carefully. Keep it for future reference. Using the product contrary to the user manual may result in serious complications.

ATTENTION! The product is intended for use by the patient and the guardian only on a flat, hard surface.

ATTENTION! The role of a guardian may be performed only by an adult, physically and mentally fit.

ATTENTION! During the use of the product, the guardian may not be under the influence of any intoxicants or medications that may affect his efficiency, orientation and coordination.

ATTENTION! If any disturbing symptoms appear during the use of the product (e.g. fainting, weakness) of the guardian or user, stop using the product immediately and safely remove the user from it. Each time such a case should be reported to the attending physician or physiotherapist.

ATTENTION! Using the product on the stairs is prohibited!

ATTENTION! For the safety of the guardian and user, check the product each time before use. Make sure that the product does not have any damage, tears, abrasions in belts, seams and other soft elements, and that the rigid elements of the product are not damaged. Do not use an incomplete or damaged product! This can seriously injure both the guardian and the user. Only an undamaged product and its proper use guarantee safe functioning of the **walk^{er}** orthosis!

ATTENTION! Before each use, check that all user's straps are properly fastened. Make sure that they do not unfasten under the influence of weight. Improper attachment and stabilization of the user may cause the user to slip out, fall and seriously injure the user!

ATTENTION! Do not use the product near fire. This is dangerous when using the product and can seriously injure the user and guardian.

ATTENTION! The product cannot be modified independently. Each intervention (sewing on elements, fastening, unfastening) and using the product contrary to the instructions for use increases the risk of losing health to the guardian and user.

ATTENTION! Adjust the circumferential ranges with Velcro, straps and buckles with the user only in a lying position. Do not adjust the fasteners while standing! It may fall and seriously injure the user and guardian!

ATTENTION! When adjusting the product, follow the instructions given in the "Instructions for use" section of this manual (chapter 5). The product should always be put on and taken off in a lying position.

ATTENTION! Keep the product clean, following the maintenance and hygiene rules contained in chapter 4 of the instruction. Cleaning the product should be carried out systematically, ensuring that it is kept in proper condition. Do not clean the product while it is in use.

ATTENTION! Make sure the hip belt is secured at the height of the hip girdle. Fixing at the level of the lumbar spine may result in serious pain consequences.

ATTENTION! The user should only be stabilized in shoes. High heels, wedges, flip-flops, sandals and other similar footwear should not be used while using the product.

4. SAFETY RULES

ATTENTION! The straps of the product cannot be fastened too loosely. They also cannot be twisted or rolled up.

ATTENTION! Before each use of the product, make sure that it is properly adjusted to the user. Make sure the straps do not rub the skin or put too much pressure on any part of the user's body. The user should feel comfortable wearing the product and be able to breathe freely.

ATTENTION! The product should not come into contact with skin. If there are abrasions / imprints / redness on the user's body, check that the product has not been fastened too tightly. If light loosening the straps will not help, discontinue the use immediately and consult your physician or physical therapist.

ATTENTION! The manufacturer is responsible for the proper functioning of the device only if it is purchased from an authorized distributor or a specialized medical store.

ATTENTION! Do not exceed the maximum load of the product!

ATTENTION! The manufacturer is responsible only for hidden defects or damages caused during proper usage as a result of a manufacturing defect of the product.

ATTENTION! Do not use the product in a manner inconsistent with its intended use. Using the product contrary to the rules of the instructions for use may result in serious complications.

5. MAINTENANCE AND HYGIENE

Soft elements of the product, i.e. chest belt, hip belt, internal lining of the orthosis,

straps:

- hand wash in warm water with soap at 30°C,
- rinse thoroughly,
- do not iron and do not chlorinate,
- do not dry clean and do not tumble dry,
- do not storage in humid places,
- avoid high temperature and dry away from heat sources.

Sandals, rigid parts of the product:

- do not wash and rinse,
- wipe with a damp cloth,
- use soft cleansing detergents, preferably soap.

6. SETTING UP

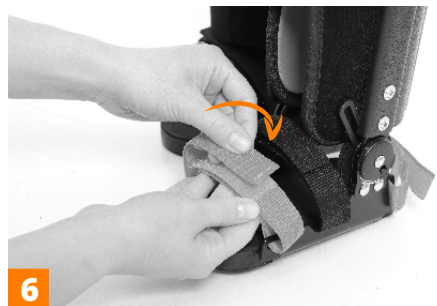
The user is placed in the orthosis in a lying position.



In order to place the user in the orthosis, unfasten all stabilizing straps so that the user can be positioned as freely as possible in the orthosis, and his individual parts of the body (pelvis, torso, chest, knees, feet) are in the right places.

Then, make sure all seat belts and straps are properly fastened.


SANDALS' STRAPS ADJUSTMENT

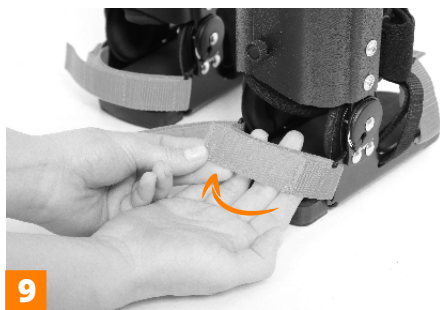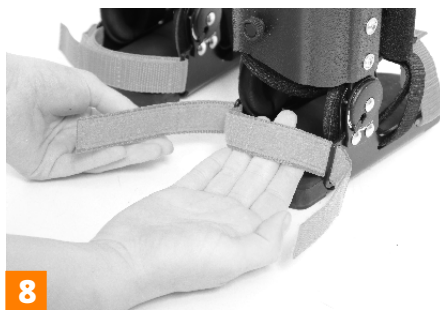
The length of the straps can be freely adjusted. The straps have vertical stitching. If you want to shorten the strap, cut off its excess before the appropriate stitching.

SANDALS ADJUSTMENT


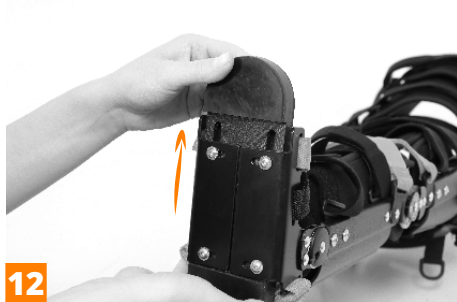
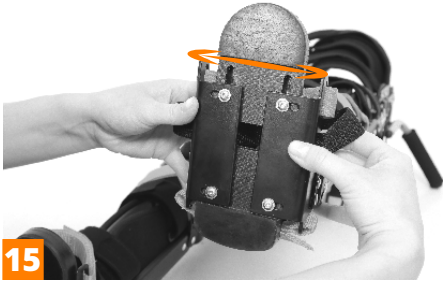
ATTENTION! Each time after setting the appropriate adjustment, tighten the screws sufficiently, checking if they are properly tightened.

CALF PADS AND STRAPS ADJUSTMENT

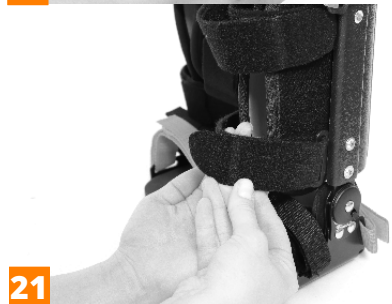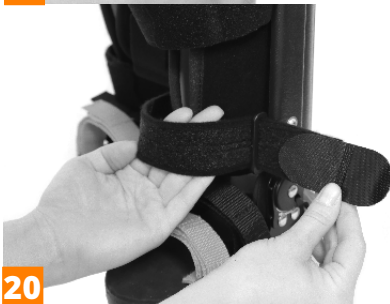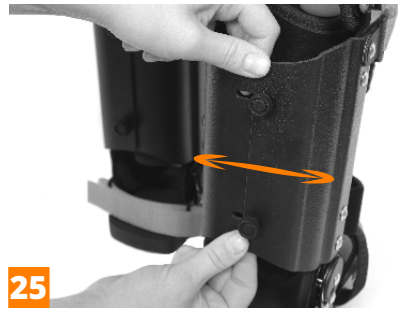ATTENTION! The straps can be cut to the appropriate length.

ATTENTION! When adjusting the calf pads, pay special attention to correctly locking them in the appropriate position.

Inadequate tightening of the knobs can result in increased pressure and cause discomfort and abrasions.

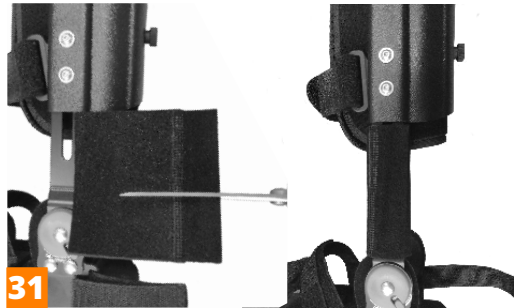
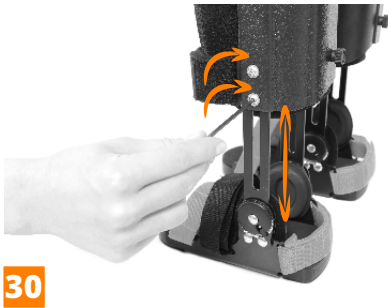
HEIGHT ADJUSTMENT OF THE CALF PADS

ATTENTION! The screws must be loosened on both sides of the orthosis.

It is recommended to use covers of the orthosis elements (fig. 31). Covers can be cut to any length.

KNEE PADS ADJUSTMENT

ATTENTION! The soft internal lining of the orthosis is fixed with adhesive tapes. They can be easily and quickly detached for adjustment or cleaning.

THIGH PADS AND STRAPS ADJUSTMENT

35

36

37

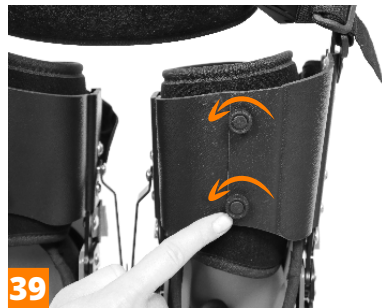
38

ATTENTION! When adjusting the thigh pads, pay special attention to correctly locking them in the appropriate position.

Inadequate tightening of the knobs can result in increased pressure and cause discomfort and abrasions.

39

40

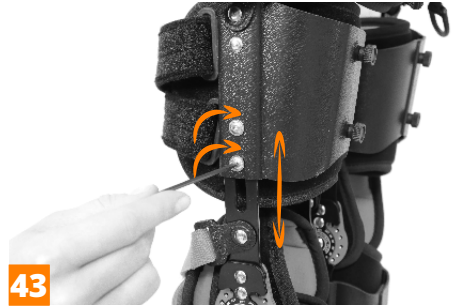
41


HEIGHT ADJUSTMENT OF THE THIGH PADS

ATTENTION! The screws must be loosened on both sides of the orthosis.

BUTTOCKS PAD ADJUSTMENT

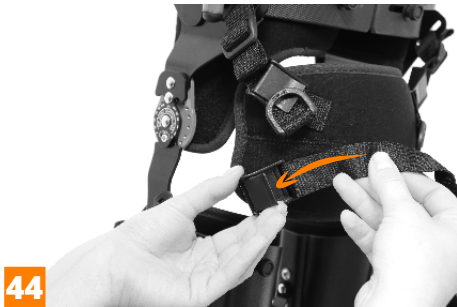HIP AND TORSO BELTS ADJUSTMENT

TORSO BELT

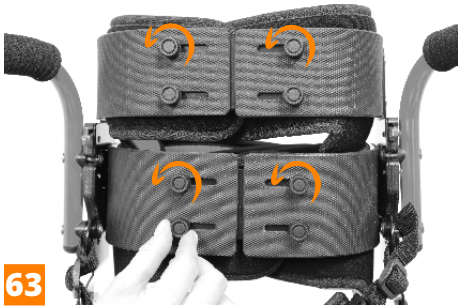
ATTENTION! The hip and torso belts have extension elements and straps (photo 54, 58, 61, 62), which can be used when the circumference of the standard belts is too small.

THE ADJUSTMENT OF THE HIP AND TORSO PART


ATTENTION! When adjusting the hip and torso element, pay special attention to properly locking them in a selected position.

Inadequate tightening of the knobs may result in increased pressure and cause discomfort and abrasions.

63

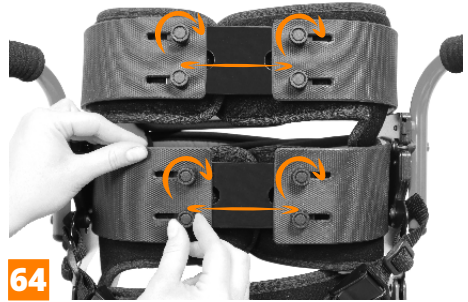64

ADJUSTING THE HEIGHT AND DEPTH OF THE HIP AND TORSO PART

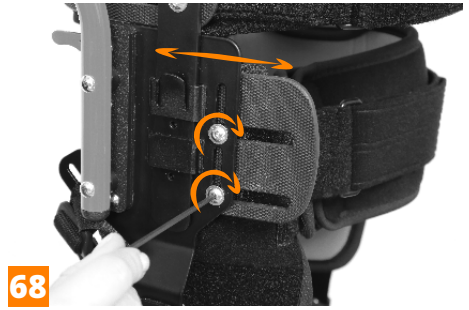
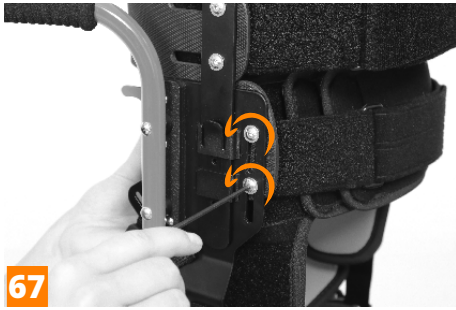
65


66


ATTENTION! It is possible to completely disassemble the torso basket. To do this, pull the basket completely out of the adjusting elements (**photo 66**).


ADJUSTING THE ROM OF THE ANKLE JOINTS

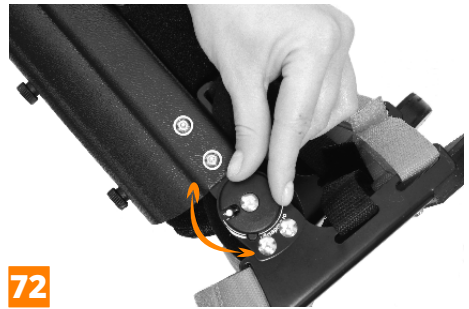75

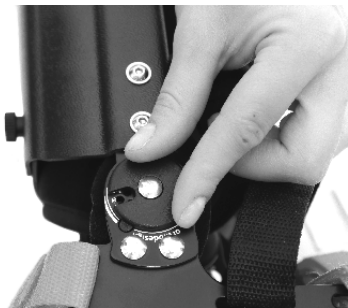76

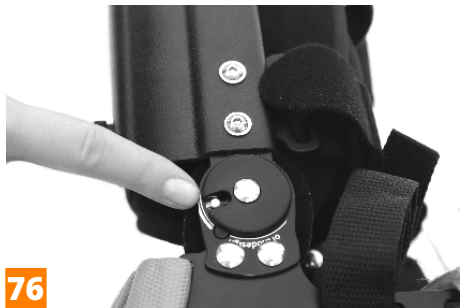ATTENTION! The heel angles are responsible for plantar flexion. Front angles are responsible for the dorsal flexion.

Remember to adjust the angle on the outer and inner sides of the feet!

77

78

79

80

81

ATTENTION! After completing the adjustment, turn the cover to the LOCK position to protect the pins from falling out.

82

ADJUSTING THE ROM OF THE KNEE AND HIP JOINTS

83

ATTENTION! The adjustment of the rom of the knee joints is carried out in the same way as in the hip joints, using the attached screws. Remember to adjust the angles on both sides of the brace!

The drop lock should always be placed in the axis of the joint.

84

85

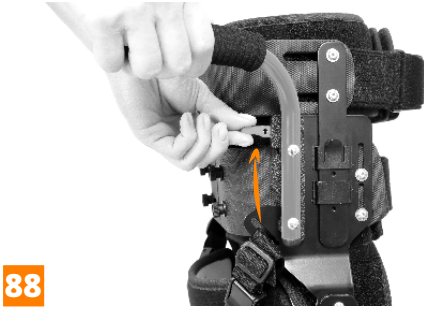
86

87

HANDLES ADJUSTMENT

88

89

It is possible to put the handles on the front, so that the user maintains eye contact with the guardian while using the product.

90

91

CLICK

The assembly of the handles is accompanied by the characteristic **CLICK** sound. Before using the product, it is absolutely necessary to check if they have been properly installed.

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